INNERVA POWER ASSISTED EXERCISE EQUIPMENT

**Operations Manual
Bedieningshandleiding
Manuel des opérations
Betriebshandbuch
Manuale operativo
Driftshåndbok
Manual de operaciones**

innerva
together in motion

V1.0 January 2025

OPERATIONS MANUAL (EN-GB)

MANUFACTURER

Shapemaster Global Ltd
(Trading as Innerva) Unit 3,
Queens Square Business Park,
Huddersfield Road, Honley,
Huddersfield, HD9 6QZ

Telephone: +44(0)1484 667474
Website: www.innerva.com
Email: info@innerva.com

Company No: 3439357
VAT No: 698476164

If your country is not listed below, please contact the manufacturer.

DISTRIBUTORS

NO/FI/DK/SE/AT/DE
Zeitz AS
Nedre Nedmarken 2,
3370 Vikersund, Norway
+4797516544
www.zeitz.com
info@zeitz.com
VAT No: 981072529

FR
SASU Silver Form Development
61 Rue Camille Pelletan, Cenon 33150,
Cidex 10, France
+33535383072
www.silver-form.com
contact@silverform-dev.fr
VAT No: FR07885059394

NL/BE/LU
BeMoreFit Solutions
W.v.d. Brinkstraat, 35 1444 HB,
Purmerend, The Netherlands
+31651624587
www.bemorefitsolutions.nl
info@bemorefitsolutions.nl
VAT No: NL860961989B01

IE/NI
Hello Goodlife Wellbeing Ireland Ltd.
38 Turlough Road, The Oaks,
Castlebar, County Mayo, F23 KC92,
Ireland
+353874828028
www.hellogoodlife.ie
info@hellogoodlife.ie
VAT No: 4184469AH

OPERATIONS MANUAL (EN-GB)

TABLE OF CONTENTS:

1. Important Safety Instructions
2. Product: Power Assisted Exercise Machines
3. Setup & Assembly
4. User instructions
5. Service and Technical Information
6. Specifications
7. Warranty

OPERATIONS MANUAL (EN-GB)

1. IMPORTANT SAFETY INSTRUCTIONS

Before using this product, it is essential to read this ENTIRE manual and ALL installation instructions.

It describes equipment setup and instructs members on how to use it correctly and safely.

⚠ CAUTION: Any changes or modifications to this equipment without prior authorisation could void the product warranty.

⚠ CAUTION: Innerva recommends seeing a physician for a complete medical exam before undertaking an exercise program. If, at any time while exercising, the user experiences faintness, dizziness, pain, or shortness of breath, they must stop immediately.

WARNING: Do not attempt to move, tip or lift the equipment whilst in motion.

WARNING: In case of an emergency requiring the equipment to stop, the emergency stop button should be engaged immediately.

DANGER: To reduce the risk of electrical shock, always unplug or switch off at a fused connector of this product before cleaning or attempting any maintenance activity.

WARNING: To reduce the risk of burns, fire, electric shock, or injury, it is imperative to connect each product to a properly earthed electrical outlet, if in any doubt seek professional advice from a qualified electrician.

WARNING: Never attempt to dismount the equipment whilst still in motion, wait until it has stopped completely, and the instruction has been displayed on the screen instructing you to do so.

WARNING: Keep a distance of at least 2ft (0.6m) around all Innerva equipment in the direction of equipment access, and in the furthest working position, keeping clear of any obstructions, including walkways, walls, furniture, and other equipment. Ensure that the equipment area is clear of potential hazards.

WARNING: Do not use equipment if the manufacturers recommendations for inspection and maintenance are not followed.

To disconnect, turn power OFF (via switch if applicable), then remove plug from electrical enclosure at the rear of the product, or store cable securely if not removable for transit.

Never operate an Innerva product if it has a damaged power cord or electrical plug, or it has been dropped or damaged. Contact your regional distributor or the manufacturer.

OPERATIONS MANUAL (EN-GB)

Make sure that the power cord is not showing signs of damage and that it is not trapped under any equipment or other objects.

If the electrical supply cord is damaged, it must be replaced by the manufacturer, an authorised service agent, or a similarly qualified person to avoid a hazard.

Always follow the on-machine display instructions and refer to the training manual or a trained member of staff for proper operation.

Do not allow other people to interfere in any way with the user or equipment whilst in use.

This appliance is not intended for use by persons (including children) with reduced physical, sensory, or mental capabilities, or lack of experience or knowledge unless they have supervision or been given instruction concerning the use of the product by a person responsible for their safety.

Keep children and animals away from the machines.

Do not use this product outdoors, near swimming pools, near water or in areas of high humidity.

Never insert objects into any opening in these products. If an object should drop inside, turn off the power, and unplug the power cord before attempting to retrieve it. If the item cannot be reached safely, contact your regional distributor or the manufacturer.

Keep all loose clothing, shoelaces, jewellery or other loose materials away from the moving parts of the equipment.

Never place liquids of any type directly on the equipment.

Do not use this product in areas where aerosol spray products are being used or where oxygen is being administered as such substances create the danger of combustion and/or explosion.

Do not use this product in bare feet.

Ensure all attachments/accessories and/or removable parts are securely fixed prior to use.

Use these products for their intended use as described in this manual. Do not use attachments on the equipment that has not been recommended by the manufacturer.

The manufacturer is not responsible for missing or damaged labels. If labels are missing, please contact your regional distributor or the manufacturer.

This is a motorised, power assisted product and will move independently of the user applying any force once activated.

OPERATIONS MANUAL (EN-GB)

This motorised power assisted product will operate at variable speeds if selected by the user.

The maximum user weight should not exceed 150kg.

Equipment must be adequately supervised by a competent person and users must have received detailed instructions on the safe use of the equipment.

Equipment should be inspected and serviced by a trained and competent person who can undertake an overall assessment of component wear/exchange, safety and any defects.

All equipment must be inspected, at least annually, by a qualified electrical engineer.

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.

OPERATIONS MANUAL (EN-GB)

2. PRODUCT: POWER ASSISTED EXERCISE MACHINES

COVERING MODELS/TYPES:

Seated user machines

- AB Pullover (ABPO)
- Tricep Dip and Leg Curl (TDLC)
- Cross Cycle (CCYC)
- Tummy Crunch (TC)
- Rotary Torso (RT)
- Side Bend Stepper (SBS)
- Chest and Legs (CNL)
- Seated Climber (SC)
- Flys and Thighs (FNT)
- Relaxer (RLXR).

Recumbent user machines

- Side Flexor (SF)
- Hipster (H)

Access user machines

- Access Tricep Dip (ATD)
- Access Rotary Torso (ART)

OPERATIONS MANUAL (EN-GB)

3. SET UP AND ASSEMBLY

1. Unpacking and Inspection

- Carefully remove all components from the packaging and check them against the order inventory list.
- Inspect all parts for visible damage. If any damage is found, do not proceed with assembly-contact your regional distributor or manufacturer for assistance.
- Dispose of packaging materials safely, keeping any user manuals or warranty information for reference.

2. Placement and Environment

- Ensure the unit is placed on a flat, stable surface to prevent instability or wobbling.
- Maintain clearance space around the unit:
 - Keep a distance of at least 2 foot (0.6 metres) around all Innerva equipment in the direction of equipment access.
- A non-slip floor surface is recommended.
- Avoid placing the equipment near:
 - Direct sunlight (to prevent overheating or material damage).
 - Water sources or high humidity areas, e.g. Swimming Pools (to avoid equipment and electrical damage).
 - Obstacles or walls that may interfere with movement.

3. Assembly Instructions

Assembly and set up of the equipment must be undertaken by the manufacturer or a trained and competent representative of the regional distributor.

Secure all fasteners and bolts ensuring they are secure, but do not overtighten.

Connect all necessary cables, ensuring they are free from twisting or entanglement and compliant with section 4: Power Connection.

Check all moving parts to ensure they operate smoothly.

OPERATIONS MANUAL (EN-GB)

4. Power Connection and Startup

Electrical Specification for Innerva PAE Equipment

For a suite of 10 or 12 units, 10 to 12 switched 5amp fused electrical spurs outlets, divided into two equal ring mains, each with a breaker of 16amp, 100ma should be provided.

Innerva machines are fitted with Transient Voltage Surge Suppressors (TVSS) which give ultra-high-speed protection, however, fitting a Type 2 ASD and a 100mA RCBO on the AC side are also encouraged to minimise electrical surges, thus avoiding irreversible damage to the low voltage touch screen and stopping the machine from functioning correctly.

Fuse Rating

The power requirements for Innerva exercise machines when no effort is applied is typically in the range of 80 to 115 watts, depending on the selected speed.

When a machine is used actively, the user assists the motion, and the power requirement can fall by up to 50% depending on the applied effort. Therefore, the current load is only a few hundred milliamperes. However, incorrect usage can occur when the user attempts to resist rather than assist. In such cases the power consumption can increase significantly. Currents have been measured in the range 2.5 – 3.0A.

Whilst it is difficult to sustain resisting the machine over an extended period it may be possible to cause a 3A safety fuse to fail. Therefore, the recommendation for safety fusing is to fit a 5A fuse.

Earth Leakage

As Innerva exercise machines utilise an AC motor driven by a variable frequency inverter, there is an earth leakage current due to the inverter in the range 3.5 – 5.0 mA. The earth leakage current is inverter output frequency dependent but always falls in the quoted range.

When several Innerva machines are wired onto the same circuit, there is a high probability that a 30mA will trip, especially as a 30mA RCD can trip with a leakage current as low as 15mA.

It is recommended that a 100mA RCD is fitted for each circuit with 5 or more Innerva machines.

Each machine should be wired either on a switched fused spur or via an IEC plug and socket conforming to BS4343 (UK) or the IEC 60309 standard.

OPERATIONS MANUAL (EN-GB)

4. USER INSTRUCTIONS

1. Powering On and System Check

- Press the power button or switch to activate the unit.
- Wait for the control panel to initialise and display default settings.

2. Adjusting the Equipment

- Seat Adjustments (Cross Cycle only):
 - Adjust the seat height and depth to ensure proper posture.
 - Lock the seat in place before use.
- Resistance and Speed Settings:
 - Select an appropriate resistance level based on personal fitness or rehabilitation needs.
 - Adjust speed gradually-avoid sudden changes to prevent strain or injury.
- Arm and Leg Positioning:
 - Maintain a comfortable, ergonomic position.
 - Use handles and supports as needed for stability.

Innerva booster cushions (supplied with the equipment) can be used by individuals of differing heights to ensure correct posture on the 'Chest & Legs' 'Seated Climber' and 'Flys & Thighs' machines. People who are exceptionally tall or small (i.e. less than 5' or taller than 6'2") are advised to be assessed by an experienced instructor to achieve an optimal position on the equipment.

3. Operating the Equipment

Users are presented with six options which include three predefined programmes and three fixed speeds. Users can select their preferred option by touching the relevant display button.

At the end of each programme a countdown screen will appear followed by instruction to dismount the machine safely.

An emergency stop button is situated at the top of the machine and shuts down the machine immediately (excluding the Relaxer).

A warning screen appears to alert staff along with audible beep sound. Please note, the emergency stop button should only be used where appropriate. It should not be used as a regular method of stopping the equipment.

Equipment users should:

- Start by moving at a moderate pace to warm up.
- Follow the recommended exercise or rehabilitation program if applicable.
- Maintain smooth, controlled movements-avoid jerking or excessive force.
- If discomfort occurs, immediately stop movement and press the emergency stop button (if fitted).

OPERATIONS MANUAL (EN-GB)

4. Post-Use Guidelines

Turn off the unit completely and unplug if necessary.

Clean and maintain after each use:

- Wipe down handles, seats, and control panels with a non-abrasive cloth.
- Avoid using harsh chemicals that could degrade materials & follow manufacturers cleaning recommendations.

5. Safety Precautions

- Read and ensure that the IMPORTANT SAFETY INSTRUCTIONS at the start of this manual are followed at all times.
- Always use proper footwear and attire to prevent slips or discomfort.
- Do not exceed the recommended usage time per session.
- Keep children and pets away from moving parts.

Stop immediately if:

- Sharp pain or discomfort occurs.
- Unusual noises or vibrations are detected.
- The unit fails to respond to controls.

OPERATIONS MANUAL (EN-GB)

5. SERVICE AND TECHNICAL INFORMATION

1. Routine Maintenance Checklist

Daily:

- Wipe down the unit to remove dirt, dust, and sweat.
- Check for any loose screws or parts before use.

Weekly:

- Inspect all moving parts (handles, pedals, seats) for smooth operation.
- Tighten bolts if needed.
- Check electrical connections and for frayed wires.

Annually:

- Schedule a professional inspection to assess & service internal components.

2. Troubleshooting Common Issues

Issue	Possible Cause	Solution
The unit won't turn on	Power supply issue	Check the plug, switch, and circuit breaker.
Display panel not working	Error whilst powering up	Power off for reset & then power back on
Strange noises during use	Loose bolts or lack of lubrication	Schedule a professional inspection to assess & service internal components.

3. When to contact Manufacturer or Regional Distributor:

- If error messages persist after restarting the unit.
- If there are broken or missing parts that cannot be replaced manually.
- If the machine stops functioning entirely despite troubleshooting.

OPERATIONS MANUAL (EN-GB)

6. SPECIFICATIONS

General Information

- Designed and Manufactured in the UK
- Power Assisted (Plug and Play Installation)
- Standard 4.3" Touch Screen Console
- Offers manual and program options
- Targeted Muscle Groups (Varies per unit)

Electrical Specification

- Voltage: 220V/240V (110V-USA)
- Amps: 3 Amps per machine
- Plug Cable Length: 1.5 meters

Upholstery & Covers

- Side Panels: High impact-resistant ABS plastic
- Covers - Flame retardant vinyl
- Resistant to bacteria and moisture (EN 1021-1&2 compliant)

Dimensions & Weights

Seated user machines

AB Pullover (ABPO)

- Dimensions: W95cm x D170cm x H130cm
- Weight: 84.8kg

Tricep Dip and Leg Curl (TDLCL)

- Dimensions: W95cm x D120cm x H132cm
- Weight: 88.4kg

Cross Cycle (CCYC)

- Dimensions: W70cm x D140cm x H130cm
- Weight: 102.4kg

Tummy Crunch (TC)

- Dimensions: W105cm x D102cm x H124cm
- Weight: 83.8kg

Rotary Torso (RT)

- Dimensions: W116cm x D118cm x H119cm
- Weight: 85.2kg

Side Bend Stepper (SBS)

- Dimensions: W114cm x L125cm x H147cm
- Weight: 100.6kg

Chest and Legs (CNL)

- Dimensions: W108cm x L130cm x H153cm
- Weight: 90kg

OPERATIONS MANUAL (EN-GB)

Seated Climber (SC)

- Dimensions: W111cm x L125cm x H172cm
- Weight: 114.8kg

Fly and Thigh (FNT)

- Dimensions: W130cm x L119cm x 153cm
- Weight: 125.4kg

Relaxer (RLXR)

- Dimensions: W94cm x L173cm x H95cm
- Weight: 51.6kg

Recumbent user machines

Side Flexor (SF)

- Dimensions: W150cm x D192cm x H70cm
- Weight: 129.4kg

Hipster (H)

- Dimensions: W88cm x D195cm x H130cm
- Weight: 95.2kg

Access user machines

Access Tricep Dip (ATD)

- Dimensions: W75cm x D125cm x H64cm
- Weight: 80kg

Access Rotary Torso (ART)

- Dimensions: W77cm x L161cm x H122cm
- Weight: 82.04kg

OPERATIONS MANUAL (EN-GB)

7. WARRANTY

For warranty claims, provide:

- Model number.
- Serial number.
- Proof of purchase.

Warranty and Service Requests:

All models come with a limited warranty covering:

- Frame and structure: 1 year.
- Electrical components: 1 year.
- Wearable parts (grips, seats, cables): 6 months.

Service requests should be submitted via:

Any service, maintenance or fault report and request must be made through our website: **<https://www.innerva.com/service-and-maintenance>** in the UK or via your regional distributor's procedure.

Refer to distributor list on page 2 for contact details.

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